

# Putting a disinfectant washer out of operation including emptying dispensing system

## Instruction:

Putting the dishwasher out of operation for a long time (greater than 3 months) with emptying dispensing system

If the machine is not to be used for an extended period of time, please carry-out the following steps.

- Remove all load from machines baskets and inserts
- Remove the filter combination and clean coarse, fine, microfine and flat filter
- Clean the non-return valve and drain pump impeller
- Re-insert the non-return valve and filters
- Clean filters at mobile units accordingly
- Take out the suction lances from process chemical containers
- Arrange suction lances upside down and in a high position to allow the liquid to run down in the direction of the machines dispensing pumps by gravity
- Activate dispensing pumps to remove process chemical from suction lances and hoses
- Use a sufficient container with Demineralized water and arrange suction lances in the container
- Activate dispensing pumps to rinse process chemicals residues from suction lances and hoses
- Remove the suction hoses from the container
- Arrange suction lances upside down and in a high position to allow the liquid to run down in the direction of the machines dispensing pumps by gravity
- Activate dispensing pumps to remove residue from suction lances and hoses
- Carefully dry the suction lances with a towel. It is recommended to cover the suction lances separately with a suitable plastic bag
- Run the Rinsing programme (cold wash cycle) This will ensure that any traces of process are dissolved and rinsed away
- Remove the filter combination and remove residual water using a syringe or sufficient sponge. Finally use a towel for proper drying
- Re-insert the filters
- Close the onsite water intake valves
- Inform the user not to close the machines for at least 12 hours
- Switch the machine off using the On/Off button at the machines control panel
- Switch the machine off at the on-site main switch / fuse breaker

## Putting the dishwasher back into operation after a period of non-use

- Open the water intake
- Switch on the on-site main switch/fuse breaker
- Open the machines door
- Check the machine and baskets/mobile units according to the list for daily routine checks
- Check all screw connections in the wash cabinet for proper tightness
- Fill the built-in chemical containers with process chemicals and connect the suction lances
- Switch the dishwasher on
- Vent/prime the dispensing system
- Run the Rinsing programme (cold wash cycle) to ensure any traces of process are dissolved
- Run a standard disinfection cleaning programme

